

## User's manual

- Name: medaSEPT latex
- Product:
  - Examination and protective latex gloves slightly powdered with modified corn starch, non-sterile, single use, fitting both left and right hand, personal protective equipment of category 1 for minimal risk
  - Available sizes: XS (5-6), S (6-7), M (7-8), L (8-9), XL (9-10)
  - The design together with the labelling is compliant with the MDD 93/42/EEC and PPE 89/686/EEC
- Transportation and storage conditions:
  - Protect from sunlight, fluorescent light and ozone
  - Store in a dry place at 5-30°C
  - Recommended relative humidity in the place of storage is 60±20%
  - Store at least 1 m away from heating devices and fire sources
  - Do not store close to solvents, oils, fuels or lubricants
  - Carton boxes of gloves are appropriate for transportation and storage
- Cleaning: single use gloves, not to be cleaned
- Disposal (utilization):
  - Used gloves may be contaminated biologically or with hazardous chemical substances
  - After use, the gloves must be disposed of pursuant to applicable provisions of local law
  - Burying or burning of the gloves must be performed in controlled conditions
- Precautions:
  - Before use, the gloves should be checked for defects or production imperfections
  - If, while using, the gloves are fractured, torn or punctured, they must be immediately replaced and in case of any doubt a new pair of gloves should be used
  - All chemical substances must be kept away from the skin even if they are not hazardous
  - In case chemical substance penetrates to the skin, it should be immediately washed off with a large amount of water and some soap
  - Avoid using gloves contaminated on the inside as this may result in irritation leading to skin inflammation or other serious injuries
  - Do not use gloves if there is a risk of the gloves being entangled by the moving machine parts
  - Gloves must not be used in contact with open fire or to protect against any type of sharp tools
  - Gloves are not designed for welding, protection against electric shock, ionizing radiation or contact with hot or cold objects. A minimum of one pair of gloves must be used per patient and per single medical procedure – disposable gloves
  - While using gloves, only mild washing detergents and disinfecting agents may be used which do not cause hand skin irritation
  - Before putting on gloves, hands should be thoroughly dried
  - After each removal of gloves, it is recommended to thoroughly wash and dry the hands as well as apply conditioning cream
  - Do not use gloves should either the user or the patient having contact with the gloves be allergic to latex
  - Natural rubber latex gloves may stimulate allergic effects including anaphylactic reactions
  - Length of protective gloves of less than 270 mm – gloves for special purposes as they are also examination gloves

- Hazardous components:
  - Some gloves may contain components which are known to be a possible cause of allergy for persons sensitive to those components who may experience irritation through contact or allergic reaction
  - Natural rubber latex gloves may stimulate allergic effects including anaphylactic reactions
  - In case of an allergic reaction, medical assistance must be sought immediately
- Final remarks:
  - Information contained in the present document is to help the user select appropriate personal protective equipment
  - Examinations and laboratory tests results are aiming at facilitating the choice, however, they do not simulate factual usage conditions
  - The responsibility for making the final decision on the use of a given personal protection equipment for intended use lies with the user
  - Product which is partly torn or damaged must be immediately withdrawn from usage as it does not ensure the protection declared by the producer and its further usage may be a cause of additional hazards
  - Gloves do not provide protection against risks not enumerated in the present document
- Responsible entity: Medasept Sp. z o. o., ul. Bałtycka 6, 61-013 Poznań